

BladeFlask EVO

100 | Scalpel Blade Removal System

How to Use

INDICATOR:
SHOWS WHICH WAY THE
SHARP EDGE MUST FACE.

THIS PART OF THE
SCALPEL HANDLE IS
CALLED THE TANG.

REMEMBER:

Sharp edge right
Estremità affilata rivolta
verso destra
Tang "UP"
Tang IN ALTO

Use with
Mounting Bracket

COUNTER

LANDING PAD

BladeFlask EVO
100 | Blade Remover

1 Place
Mettere

2 Push
Spingere

3 Pull
Tirare

REMEMBER:
Sharp edge right
Estremità affilata rivolta
verso destra
Tang "UP"
Tang IN ALTO

Use with
Mounting Bracket

Maximum
Capacity:
100 Blades

CE
PATENTED
www.clicksmart.com



EC REP

Donawa Lifescience
Piazza Albania, 10
00153 Rome, Italy

CLICKSMART®
... and be safe


Clicksmart Pty Ltd.
Level 1, 148 Boundary St, West End,
Brisbane, QLD 4101, Australia
+61-7-3844-1182 T

hello@clicksmart.com | www.clicksmart.com

MOUNT TO WALL OR TROLLEY:

1. If you haven't mounted your BladeFlask EVO before, prepare the universal mounting bracket*.

A BRACKET HAS DOUBLE SIDED
TAPE ON BOTTOM

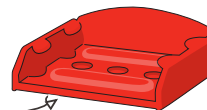


B OR USE SCREWS.



* See fitting template and instructions inside
the bracket packaging.

2. BladeFlask EVO can be mounted
on a wall or on a
trolley or bench.

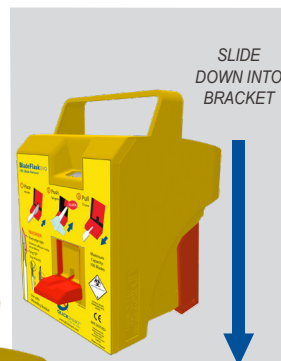

SLIDE
DOWN INTO
BRACKET






SLIDE
BACK INTO
BRACKET

NOTE: BladeFlask EVO
is to be mounted on the
Mounting Bracket during
use.

REMOVE BLADE SINGLE-HANDED ACTION:

1. Place blade on
the landing pad
with the sharp
edge facing
to the right (or
'tang' up).
2. Push up the
scalpel handle into
BladeFlask EVO
along the ramp
until you hear a
click.
3. Remove the
handle.

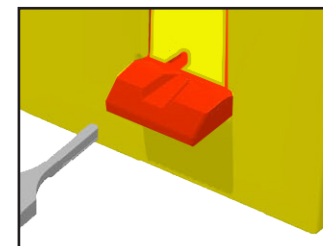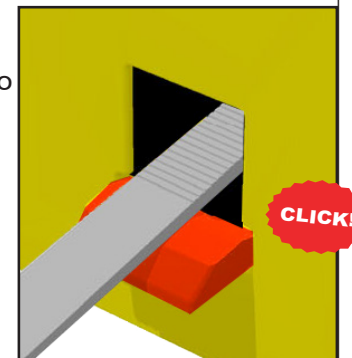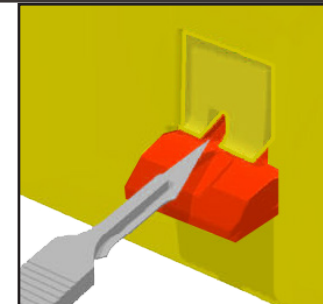

NOTE: BladeFlask EVO is disposed of as
medical waste after the counter automatically
shuts off when it reaches full capacity.

BFEIN101020A1