


How to Use

INDICATOR:
SHOWS WHICH WAY THE
SHARP EDGE MUST FACE.

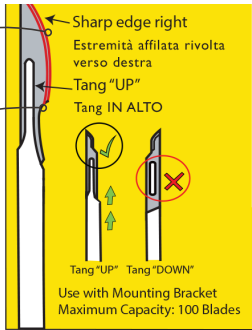

THIS PART OF THE
SCALPEL HANDLE IS
CALLED THE TANG.

COUNTER


LANDING PAD



MOUNT TO WALL OR TROLLEY:

1. If you haven't mounted your BladeFlask EVO before, prepare the universal mounting bracket*.

A BRACKET HAS DOUBLE SIDED TAPE ON BOTTOM

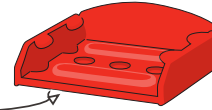


B OR USE SCREWS.



* See fitting template and instructions inside the bracket packaging.

2. BladeFlask EVO can be mounted on a wall or on a trolley or bench.

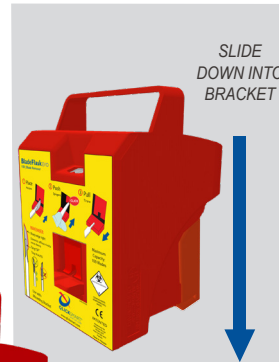


SLIDE BACK INTO BRACKET


NOTE: BladeFlask EVO is to be mounted on the Mounting Bracket during use.

REMOVE BLADE SINGLE-HANDED ACTION:


1. Place blade on the landing pad with the sharp edge facing to the right (or 'tang' up).

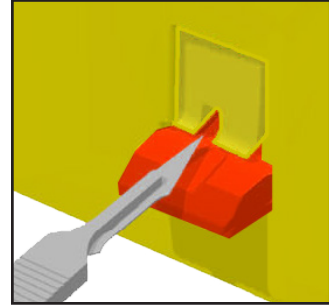

2. Push up the scalpel handle into BladeFlask EVO along the ramp until you hear a click.

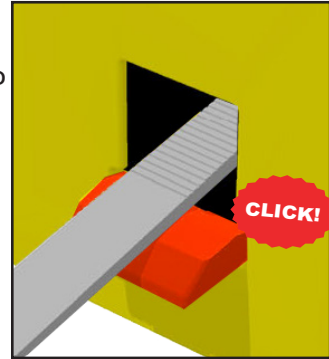

3. Remove the handle.

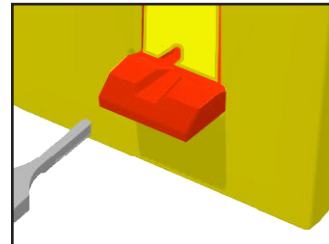

NOTE: BladeFlask EVO is disposed of as medical waste after the counter automatically shuts off when it reaches full capacity.